Adapter Set DAC Validation

AL 3300, AL 3305
AL 3309, AL 3310, AL 3311

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Table of Contents

General safety and warning information	5
Unpacking/Scope of delivery	7
Overview	11
<i>Set for the thermal test for DAC Universal MK III</i>	11
<i>Set for the thermal test for DAC Universal MK IV</i>	11
Positioning	13
<i>Thermal test</i>	13
AL 3309 - PCD test body (1)	13
Indicator in the PCD Unscrew the PCD test piece in the middle (c). Insert the indicator (Class 5 chemo-indicator suitable for DAC) and screw in the test piece again.	13
AL 3310 - Logger adapter (2)	15
<i>Preparation for the thermal test DAC Universal MK III</i>	17
Attaching the logger holder	17
Positioning the logger holder	17
<i>Preparation for the thermal test DAC Universal MK IV</i>	19
Attaching the logger holder	19
Positioning the logger holder	19
AL 3311 - Positioning the screw with universal holder	21
Cleaning and maintenance	23
<i>Disposal</i>	23
Spare parts:	23
Permissions	23
<i>General information:</i>	23

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General safety and warning information

The set is intended only for use in the DAC Universal.

Use in chemicals or in water with a conductance of $> 10\mu\text{S/cm}$ is not permitted.

The area of application extends only to validation of thermal processes and the validation of the cleaning performance in the DAC Universal.

Accessories are required for thermal testing on the DAC Universal MK IV.

No data loggers are included in the set. Data loggers of the EBI 11 series are required for the intended use:

Recommended data logger set:

SL 3301 Art. no.: 1250-3301

Thermal testing on the DAC Universal MK IV requires a center column.

AL 3304 Art. No.: 1248-3304

Three different adapters for manual and angle piece connections are included in the set (referred to as lid adapters in the following).

Always make sure that the correct lid adapter is used. The wrong adapter can lead to damage to the DAC or to the adapter.

Lightly oil in dry o-rings before attaching the test adapter (eg NitramOil 2) Do not touch the parts with bare hands immediately after a measurement. Danger of burns!

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Insert or screw in the loggers (2).
(view from below, labeling visible)

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POS a: Screw in EBI 11-T 235,
sensor pointing downwards
(chamber at the bottom,
chamber reference)

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POS b: Insert EBI 11-T 235,
sensor pointing upwards
(chamber in the middle)

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POS c: Insert EBI 11-T 236,
sensor pointing upwards
(chamber at the top)

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POS d: Insert EBI 11-P 111,
sensor pointing upwards
(chamber pressure)

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**The data loggers must be
programmed prior to positioning.**

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Engage the adapter for the data logger in the indicator holder groove (a) on the DAC lid in such a way that the sensor of the screwed-in logger points downwards (b).

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Turn the logger holder in such a way that the open side points towards connection 4 (c) on the lid.

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Turn the logger holder in such a way that the loggers are between the plug connections (d).

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The adapter snaps into place on the lid.

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Preparation for the thermal test DAC Universal MK IV

Attaching the logger holder

Hand-tighten the center column into the blue lid. for this, the lid has a thread in the middle (a). Insert the adapter with the data loggers into the groove (b) on the center column (c) so that the sensor of the screwed-in logger points downwards (d). The SW 7 (e) mount is intended only for releasing the center column.

Positioning the logger holder

Turn the logger holder in such a way that the open side points towards connection 1 (c) on the lid.

Turn the logger holder in such a way that the loggers are between the plug connections.

Now push the PCD onto connection 4.

The ISO adapter snaps into place on the lid. To attach Sirona or KaVo adapters on the lid, the knurled screws supplied must be screwed in.

Tighten the screws by hand.

Ensure that everything is firmly fixed on the lid.

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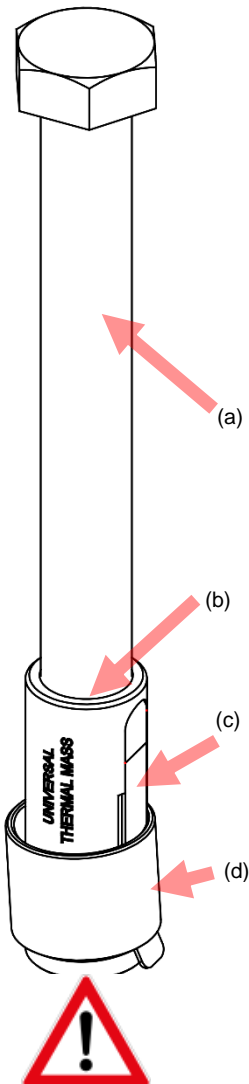
AL 3311 - Positioning the screw with universal holder

Mount the screw (a), screwed together with the universal holder (b), onto the plug connection.

The universal adapter locks on the connection for the connection. It may be necessary to slightly lift the spring (c) when putting it on.

ISO Connection:

A fixation with locking ring (d) isn't necessary.

Sirona Connexion:

Slide the locking ring (d) over the retaining spring to fix the screw with bracket

KAVO Connection:

Keep a distance of approx. 1 mm from the lid with the screw and push the locking ring (d) over the retaining spring to fix the screw.

Equip the five free slots with the test screws.

Now the lid can be inserted into the device and the test started. If you do not have one of the appropriate ports at one or more slots, use customer instruments as a thermal load.

Ensure that everything is firmly fixed in place on the lid.

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Cleaning and maintenance

Only clean the set with a slightly damp cloth.

Do not use solvents such as acetone.

Disposal

If the set has become useless, it must be disposed of properly and in an environmentally friendly manner.

Never dispose of the set in normal household waste, but return it to the manufacturer.

Spare parts:

Spare parts are available from our Service

Permissions

General information:

Modifications of any type to the unit will invalidate the operating permit.

Information, operating manuals and declarations of conformity can be found at www.ebro.com.

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Service-Adresse / Service Address / Adress du Service

Xylem Analytics Germany Sales GmbH & Co. KG
ebro
Am Achalaich 11
82362 Weilheim
Germany

Phone: +49.(0)841.954.78.0
Fax: +49.(0)841.954.78.80
Internet: www.ebro.com
E-Mail: ebro@xylem.com

.....

Hersteller / Producer / Fabricant

Xylem Analytics Germany GmbH
Am Achalaich 11
82362 Weilheim
Germany