Hohlkörper-Adapter-Set
Hollow Adapter Set
Kit Adaptateur

AL 126

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Table of Contents

Overview	5
Unpacking/What is Included in the Delivery	5
Safety instructions.....	7
Application	9
Application with data loggers of the EBI 11 series	9
Application with data loggers of the EBI 12 series	11
What to do, if.....	13
Cleaning and Maintenance.....	13
Disposal.....	13
Technical Data.....	15

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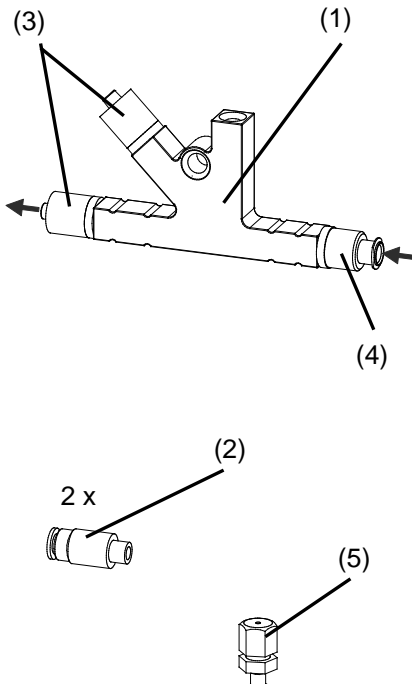
Overview

The Hollow Adapter Set

The set is designed for measuring temperature and pressure within a tube system, e. g. in endoscopes.

You need a data logger to use the hollow adapter set. (not included in the delivery)

Unpacking/What is Included in the Delivery

After receiving the delivery, please check the packaging and the content for completeness and integrity.

Following parts are included in the delivery:

- hollow adapter Y-piece (1)
- tube connection PTFE 4 mm (2)
- Luer-Lock connection male (3)
- Luer-Lock connection female (4)
- Adapter for 1.5 mm bendable temperature probe (5)
- This manual

Furthermore, there can be included different data loggers or accessories.

If you have any complaints about the AL126, please contact us. You can find our contact information on the back cover of this manual.

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Application with data loggers of the EBI 12 series

To use the hollow adapter set with the EBI 12 you will need:

(not included in the delivery)

You can use different data loggers for this.

Example:

1 x EBI 12-TP222 1340-6653
Temperature-/Pressure Data Logger
with Luer-Lock and bendable
temperature probe Ø 1.5 mm

After programming the data logger in the usual way (software necessary), sturdily rotate the logger to the Luer-Lock. (1)

Screw the adapter for the 1.5 mm bendable temperature probe sturdily in the M5 thread. (2)

Now place the sensor (3) carefully up to the stop into the adapter. Screw the adapter sturdily.

Please notice the bend radius for bendable sensors in the manual of the data loggers.

Now attach the adapter between the engines connection and the hollow system, which has to be checked.

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... no pressure or false values are shown

-the direction of flow is wrong

-the adapter is placed at the wrong position

-the hollow body or the adapter are blocked

... a wrong temperature is shown

-the sensor is not completely screwed in or inserted

- the hollow body or the adapter are blocked

...

Cleaning and Maintenance

Clean the adapter and the connections with a damp cloth.

Do not use solvents, e.g. acetone, because this damages the material.

Disposal

If the device becomes no longer fit for purpose, it must be disposed of in a suitable, environmentally-friendly manner.

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Technical Data

Operating temperature 0 to +125°C
Storage -40 to +70°C

Dimensions (without connections)

(L x W x H) 60 x 8 x 34 mm
Flow channel Ø 2 mm
Material Adapter PEEK
Connections
Luer-Lock M5 thread, male
M5 thread, female
Tube M5 thread
Externally Ø 4 mm
Internally Ø 2 mm

Material connections stainless steel
Weight appr. 37 g

Changes or modifications of the device could void the user's authority to operate the equipment.

Users manual or instruction manual may downloaded at www.ebro.com over the internet.

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Service-Adresse / Service Address / Adress du Service

Xylem Analytics Germany Sales GmbH & Co. KG
ebro
Dr.-Karl-Slevogt-Str. 1
82362 Weilheim
Germany

Phone: +49.(0)841.954.78.0
Fax: +49.(0)841.954.78.80
Internet: www.ebro.com
E-Mail: ebro@xyleminc.com

.....

Hersteller / Producer / Fabricant

Xylem Analytics Germany GmbH
Dr.-Karl-Slevogt-Str. 1
82362 Weilheim
Germany